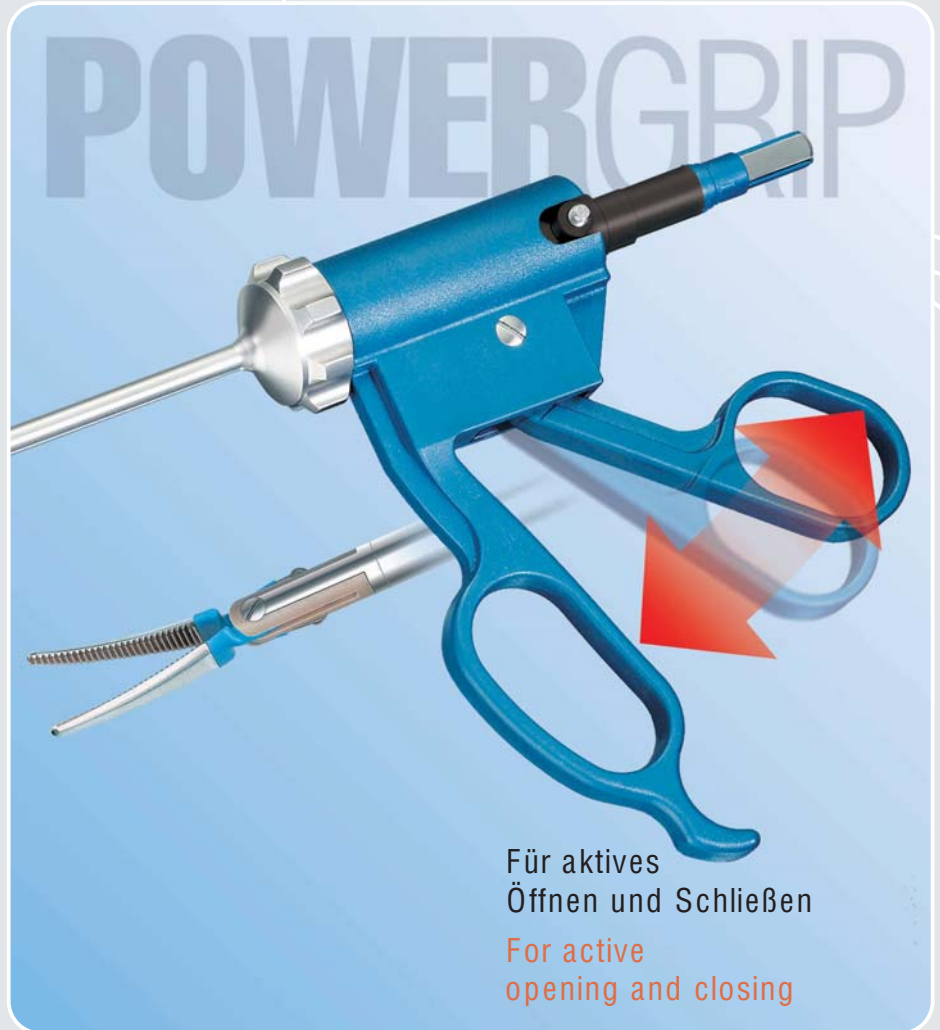
bissinger

POWERGRIP Bipolare Koagulationszange
POWERGRIP Bipolar Coagulation Forceps

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For active
opening and closing

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Precise coagulation, more powerful grasping, better cutting and more exact work: the Bissinger POWERGRIP

The Bissinger POWERGRIP is an instrument used for grasping, cutting and bipolar coagulation in Minimally Invasive Surgery. It is characterized by a newly developed mechanism enabling well adjustable opening and closing of the jaws with very high pressure while grasping and cutting. This is accompanied by precise coagulation of the tissue. The jaws are not moved in the customary way by spring power, but by means of a double hinge which is connected electrically and mechanically to the handle by a slide rod.

This new principle allows extended application of minimally invasive bipolar coagulation, and represents precision mechanics in the truest sense of the word!

The connector for standardized bipolar cables is integrated in the particularly ergonomic handle made of high quality PEEK polymer. This saves the purchase of expensive special cables and prevents them from being connected incorrectly. We supply standardized bipolar cables for connection to practically all common high-frequency generators.

Beyond that, the Bissinger POWERGRIP offers a number of important further advantages:

- exchangeable electrode inserts for different applications
- easy handling
- excellent cleaning and ease of repair

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The instrument jaws are insulated up to the end of the grasping or cutting zone, in order to avoid unintentional coagulation.

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Quick exchange of electrode inserts

The different inserts can be exchanged in only a few seconds: The electrode is screwed into the shaft, inserted in the handle and locked in place by means of a click-stop. At the same time the connection to the integrated electric connector is established.

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The star-shaped wheel on the shaft allows the complete electrode insert to be rotated up to 360° with the forefinger. This ensures that the instrument jaw can be adjusted to any position desired. The fact that the position of the jaw is maintained when open or closed ensures precise work when dissecting, cutting, grasping and coagulating.

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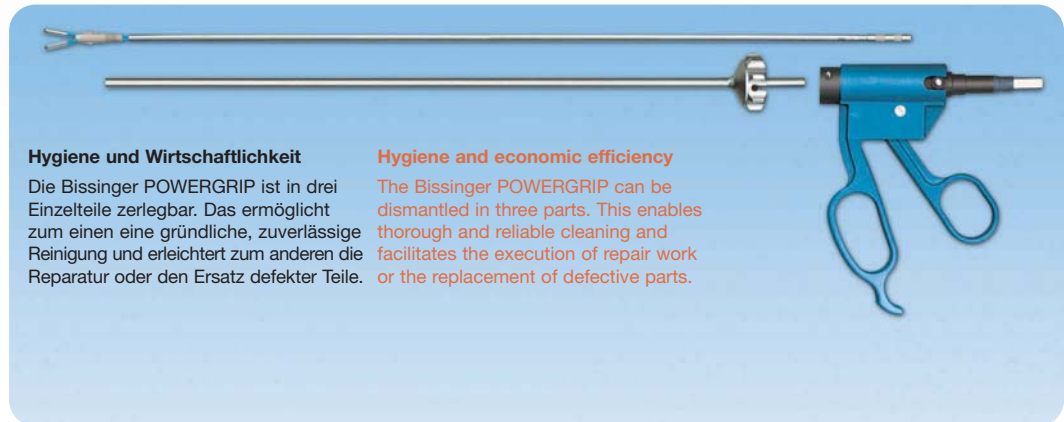
Hygiene and economic efficiency

The Bissinger POWERGRIP can be dismantled in three parts. This enables thorough and reliable cleaning and facilitates the execution of repair work or the replacement of defective parts.

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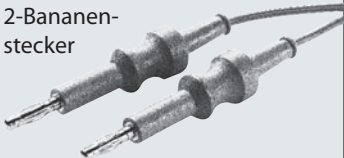
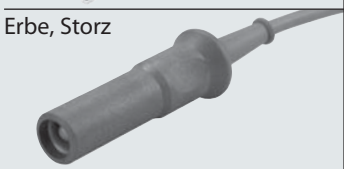
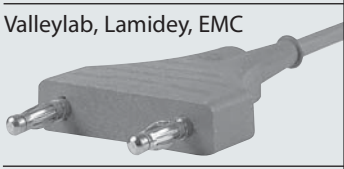
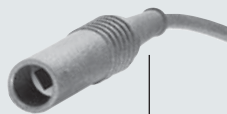
Our Products and Services

- Surgical instruments
- Bipolar and monopolar instruments
- Accessories for Electrosurgery
- Development of instruments to customer specifications
- Service and repair

Bissinger – Focus on better Solutions

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Medizintechnik GmbH